



Developed to Save: PATIENTS ♦ STAFF ♦ TIME ♦ MONEY


aXtraHand LLC  
51528 Industrial Drive  
New Baltimore, MI 48047  
P - 586-777-0018  
F - 866-777-0535  
www.axtrahand.com

## Instructions for: **Wheelchair to IV Pole Clamping System (1" to 2")**

**Product Code: CH-7000** (Patent Pending)

### Attaching to the Wheelchair

1. Cut off the end of the plastic handle on the wheelchair to allow aXtraHand CH-7000 Clamping System to be attached  
NOTE: you can use either the left or the right side, whichever you prefer.
2. Remove the bolt with a 5/32 allen wrench
3. Install locking bolt into the wheelchair – make sure the “flats” on the locking bolt are in a vertical position to allow proper alignment for clamping onto the IV Pole. Tighten the locking bolt with a 3/16 allen wrench
4. Assemble arm to locking bolt – reinstall bolt from step 2 and tighten with a 5/32 allen wrench
5. Check alignment with IV Pole. If further adjustments are needed, follow steps 2 through 5 – if you have any questions, please contact us at 586-777-0018.

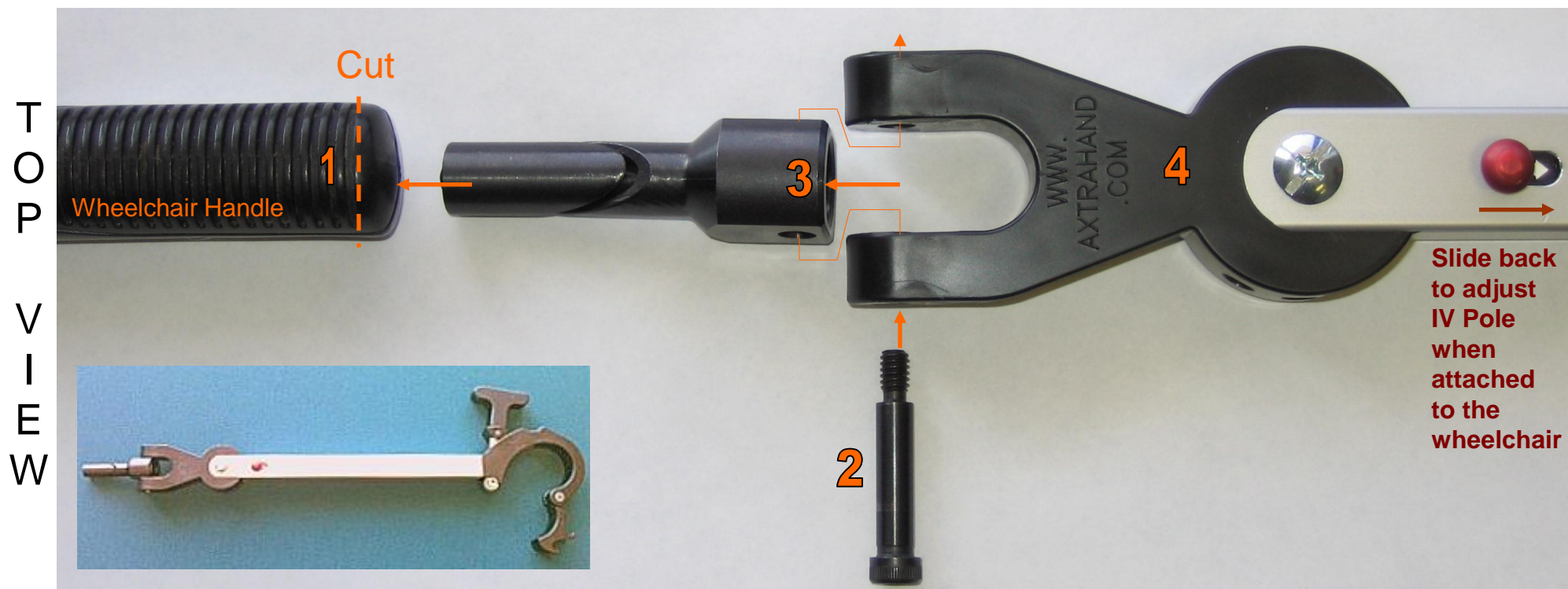

\*\*Due to different designs of wheelchairs, you may experience difficulties with installing the locking bolt into the wheelchair handle. If this occurs, please contact us at 586-777-0018.



Developed to Save: PATIENTS ♦ STAFF ♦ TIME ♦ MONEY


aXtraHand LLC  
51528 Industrial Drive  
New Baltimore, MI 48047  
P - 586-777-0018  
F - 866-777-0535  
www.axtrahand.com

## Instructions for: **Wheelchair to IV Pole Clamping System (1" to 2")**

Product Code: **CH-7000** (Patent Pending)

### Attaching to the IV Pole:

The **CH-7000 HDB Clamp** wraps around the entire range of square or round bars, from 2" OD down to 1" OD, all without marring the equipment. The unique 2 position latch on the link saves time when switching from a 2" bar to a small bar, just connect to the second latch for the smaller sizes.

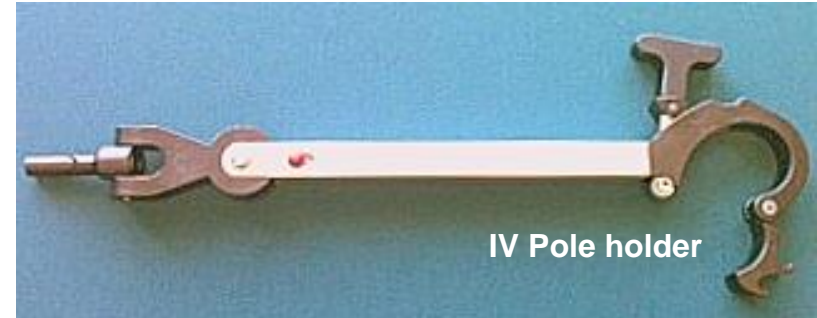

The quick release handle takes only 2 turns to be able to drop the locking mechanism, and release the clamp from the bar.

### Storage & out-of-the-way positions for CH-7000

The CH-7000 Clamping System can be easily manipulated into 9 locking positions. Two of the storage/out-of-the-way positions are:

90° = for easy maneuverability while transporting a patient w/no IV (please see photo below)

Vertical = for easy storage (please see photo below)

